## **APPLE VISION BRAND CONTACT LENS-CARE PRODUCTS**

1	AIRE
_	CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION
2	IGEL PLUS
	MULTIPURPOSE DISINFECTING SOLUTION
3	ENCORE
	ALL-IN-ONE SOLUTION
4	AIRE
_	CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION

## PATIENT PRODUCT INSTRUCTIONS FOR USE & SAFETY INFORMATION

PLEASE READ THE FOLLOWING INFORMATION CAREFULLY AND KEEP IT FOR REFERENCE.

1	AIRE CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION are
	manufactured in an ISO 13485 certified facility. These medical devices are manufactured
	under strict GMP conditions and guarantee our customers total quality.
	IGEL PLUS MULTIPURPOSE DISINFECTING SOLUTION are manufactured in an ISO 13485
2	certified facility. These medical devices are manufactured under strict GMP conditions and
	guarantee our customers total quality.
	<b>ENCORE</b> ALL-IN-ONE SOLUTION are manufactured in an ISO 13485 certified facility. These
3	medical devices are manufactured under strict GMP conditions and guarantee our customers
	total quality.
	AIRE CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION are
4	manufactured in an ISO 13485 certified facility. These medical devices are manufactured
	under strict GMP conditions and guarantee our customers total quality.

## **PRODUCT DESCRIPTION**

1	AIRE CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION is a sterile,
	aqueous, isotonic solution containing Sodium & Potassium Chloride, Disodium Edentate,
	Polyhexanide, Poloxamer, HPMC, Sodium Hyaluronate and Sodium Phosphate Buffer.
2	IGEL PLUS MULTIPURPOSE SOLUTION is a sterile, aqueous, isotonic solution containing
	Sodium & Potassium Chloride, Disodium Edetate & Power Packed Combination of 4
	ingredients which combines Polyhexanide, Poloxamer, Hypromellose & Sodium Phosphate
	Buffer.
3	<b>ENCORE</b> ALL-IN-ONE SOLUTION is a sterile, aqueous, isotonic solution containing Sodium &
	Potassium Chloride, Disodium Edetate & Power Packed Combination of 4 ingredients which
	combines Polyhexanide, Poloxamer, Hypromellose & Sodium Phosphate Buffer.

AIRE CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION contain
Hypromellose, Disodium Edetate, Sodium Chloride, Potassium Chloride, Poloxamer,
Polyhexanide, Boric Acid, Borax and Chlorhexidine Gluconate.

#### **ACTIONS**

1	AIRE CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION is a combination of special agents, that cleans your lenses better, removes protein more effectively, makes your lenses germ free and keeps them double moist so that they are more comfortable on your eyes, pH & isotonicity are adjusted to lacrimal fluid.				
2	<b>IGEL PLUS</b> MULTIPURPOSE SOLUTION is a combination of special agents, that cleans your lenses better, removes protein more effectively, makes your lenses germ free and keeps them moist & well lubricated so that they are more comfortable on your eyes, pH & isotonicity are adjusted to lacrimal fluid.				
3	<b>ENCORE</b> ALL-IN-ONE SOLUTION is a combination of special agents, that cleans your lenses better, removes protein more effectively, makes your lenses germ free and keeps them moist & well lubricated so that they are more comfortable on your eyes, pH & isotonicity are adjusted to lacrimal fluid.				
4	AIRE CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION is a combination of special agents, that cleans your lenses better, removes protein more effectively, makes your lenses germ free and keeps them moist & well lubricated so that they are more comfortable on your eyes, pH & isotonicity are adjusted to lacrimal fluid.				

### **SPECIAL ADVANTAGES**

S	DIUM HYALURONATE

Forms a lubricating shield to keep in moisture for long lasting eye comfort.

# НРМС

Forms a lubricating shield to keep in moisture for long lasting eye comfort.

# POLYHEXANIDE

A disinfectant and preservative with powerful biocidal efficacy.

# CHLORHEXIDINE GLUCONATE

A disinfectant and preservative with powerful biocidal efficacy.

# POLOXAMER

A gentle surfactant and cleaning agent. Removes protein, dirt and debris.

## SODIUM PHOSPHATE BUFFER

Keeps solution pH balanced and enhances performance of added electrolytes (sodium and potassium chloride) for more natural lens wear.

#### **INTENDED FOR USE**

When use as directed, **AIRE** CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION/**IGEL PLUS** MULTIPURPOSE DISINFECTING SOLUTION/**ENCORE** ALL-IN-ONE SOLUTION cleans, rinses, disinfects, lubricates, moisturizes, removes protein and stores contact lenses. The solution is indicated for **all soft contact lenses under Daily Wear and Frequent Replacement**. The packaging includes a lens case for storage of contact lenses during chemical disinfection.

When use as directed, **AIRE** CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION cleans, rinses, disinfects, lubricates, moisturizes, removes protein and stores contact lenses. The solution is **only indicated for rigid gas permeable and hard contact lenses** and strictly not to be used on soft contact lenses. The packaging includes a lens case for storage of contact lenses during chemical disinfection.

#### **COMPLIANCE AND HYGIENE**

- Customers are urged to follow the recommendations for Clean and Safe Contact Lens Wear.
- Carefully and regularly clean contact lenses, as directed by an Optometrist / Lens Care
   Practitioner and the patient information included with the lens care product.
- Wash and dry hands before putting on or removing lenses.
- Do not wear lenses beyond the recommended period.
- Store lenses in the proper case and replace the lens storage case at least every three months.
   Clean the contact lens case after each use with Contact Lens Care Solution and keep the case open and air dry between cleanings.
- Only fresh solution should be used to clean and disinfect contact lenses. Never re-use old solution. Contact lens care solution must be changed every day, even if the lenses are not use daily.
- Remove and stop using lenses if you experience unusual eye symptoms; seek medical consultation if necessary.

#### **CONTRAINDICATIONS (REASONS NOT TO USE)**

There are no known contraindications for use of the products as listed above. If you are allergic to any ingredient in this product, please do not use.

## **DIRECTIONS FOR USE**

## A) CLEANING / DISINFECTING / STORING

- 1. Always wash your hands and dry them with a clean towel before handling your lenses. Always start with the same eye to avoid confusion.
- 2. Open the lens case, remove the lens and place it on the palm of your hand.

- 3. Apply at least 3 drops of the solution to each lens surface.
- 4. Gently rub the lens using a backward and forward motion for 10 seconds per side.
- 5. Rinse both lens surfaces thoroughly with the solution while rubbing the lens gently between the thumb and forefinger.
- 6. Place the lens in the lens case and fill the well with the solution.
- 7. Replace the cap and screw tight. Repeat the procedure with the other eye's lens.
- 8. Allow your lenses to soak for at least 4 hours. Your lenses are now clean and disinfected and may be inserted directly into your eye.

### **B) INSERTING YOUR LENSES**

- 1. Make sure your hands are washed with a mild soap, rinsed thoroughly and dried with a lint-free towel before inserting your lenses.
- Remove the right eye's lens from the storage case. Clean and rinse the lens with AIRE CLEAR,
  MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION/IGEL PLUS MULTIPURPOSE
  SOLUTION/ENCORE ALL-IN-ONE SOLUTION/AIRE CONDITIONING & CLEANING MULTIPURPOSE
  DISINFECTING SOLUTION.
- 3. Place the lens on the tip of your right-hand index finger.
- 4. Place the middle finger of the same hand close to your lower eyelashes and hold down the lower lid. Pull upper eyelid upwards with the middle finger of the left hand.
- 5. Look straight ahead and gently place the lens on your eye.
- 6. Gently release the lid and blink. The lens will center itself automatically.
- 7. Repeat procedure on the left eye's lens.
- 8. Use a mirror if necessary.
- 9. Check to be sure that the lens is centered by covering the other eye. If your vision is crisp and clear then the lens is in place.

#### C) REMOVING YOUR LENSES

- 1. Remove cosmetics and make-up only after you have removed the lens.
- 2. Please ensure your hands are washed, rinsed and dried.
- 3. Begin procedure with the right eye.
- 4. Pull the lower lid down with the middle finger of your right hand.
- 5. Look upward, slide lens to lower part of the eye with your forefinger.
- 6. Gently pinch the lens between the thumb and forefinger and remove from the eye.
- 7. Clean and rinse your lens with **AIRE** CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION/**IGEL PLUS** MULTIPURPOSE SOLUTION/**ECNORE** ALL-IN-ONE SOLUTION/**AIRE** CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION and put them back into your lens storage case before removing lens from the other eye.

#### **HEALTHY RELATIONSHIP WITH YOUR CONTACT LENS**

- 1. Avoid strong, oily soaps while washing your hands. Make sure you rinse and dry your hands before handling your lenses.
- 2. Make sure your surroundings are smooth and clean. If you are using a basin/sink, make sure the plug is in to ensure that the lens does not get drained away.
- 3. Always clean and disinfect lenses before re-insertion into your eyes.
- 4. Always use fresh solution from the bottle. Never re-use solution left in the lens storage case.
- 5. Do not store or rinse lenses with tap water.
- 6. Use only the lens case provided to store your lenses.
- 7. Make sure the lens is fully immersed in the solution. Never use lenses unless fully hydrated (wet) in solution.
- 8. Always keep your lens case clean. Wash and rinse lens case with **AIRE** CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION/**IGEL PLUS** MULTIPURPOSE SOLUTION/**ENCORE** ALL-IN-ONE SOLUTION and air dry every day.
- 9. Check before insertion that the lens is not turned inside-out.
- 10. When inserting your lenses, always start with the same eye to avoid confusion.
- 11. The lens can never be lost in your eye. Use a well-lit mirror to help locate your lens, it will probably be on the inside of your eyelid.
- 12. Do not re-center the lens on the cornea with your finger on the lens. Instead use your eyelid to move and reposition the lens.
- 13. Avoid touching lenses with your finger nails. Make sure your finger nails are short and smooth.
- 14. If the lens is nicked or torn, do not wear it.
- 15. Avoid rubbing your eyes after you have inserted lenses. This can irritate your eyes or dislodge the lenses.
- 16. Do not allow lotion, soaps, creams, deodorants, hair sprays and cosmetics (especially metallic powders) to come in contact with the lenses. They can cause an eye infection or irritation.
- 17. Apply make-up only after inserting your lenses.

#### **MORE SAFETY INFORMATION**

- 1. Avoid eye make-up like eye liners as these can stain your lens.
- 2. Do not wear lenses overnight.
- 3. Avoid wearing lenses around fumes, irritating vapors, smoky or dusty conditions. This may irritate your eyes and/or damage the lenses.
- 4. Always wear swimming goggles while swimming with your lenses on.
- 5. On holidays, carry ample supply of solution. Make sure you have a spare pair of contact lenses.
- 6. Visit your eye-care practitioner regularly for a routine check-up.

#### **MARKET AVAILABILITY**

1	AIRE CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION 360 ml with a			
	Lens Case included.			
2	AIRE CLEAR, MOIST & REFRESHING MULTIPURPOSE DISINFECTING SOLUTION 60 ml with a			
	Lens Case included.			
3	IGEL PLUS MULTIPURPOSE SOLUTION 500 ml with a Lens Case included.			
4	IGEL PLUS MULTIPURPOSE SOLUTION 60 ml with a Lens Case included.			
5	ENCORE ALL-IN-ONE SOLUTION 360 ml *			
	This packing only comes in the form of "The ENCORE 3 in 1 DEAL!" which consists of:			
	1. 1 bottle of <b>ENCORE</b> ALL-IN-ONE SOLUTION 360 ml			
	2. 1 bottle of <b>ENCORE</b> ALL-IN-ONE SOLUTION 60 ml			
	3. 1 bottle of <b>ENCORE</b> COMFORT DROPS 10 ml			
	4. 1 Lens Case included			
6	ENCORE ALL-IN-ONE SOLUTION 60 ml with a Lens Case included.			
7	AIRE CONDITIONING & CLEANING MULTIPURPOSE DISINFECTING SOLUTION 120 ml.			

#### **MANUFACTURED BY**

STERICON PHARMA PTE. LTD.

No.9R, Sub Layout of Plot No.9, 1<sup>st</sup> Phase, Bommasandra Industrial Area, Bangalore – 560099 India An ISO 13485:2003 Certified Company

#### **MANUFACTURED FOR & DISTRIBUTED BY**

APPLE VISION SDN. BHD.

No.23, Jalan 6/33B, Batu 6 ½ Jalan Kepong, 52000 Kuala Lumpur, Malaysia.

www.apple-vision.com

EC	REP	MYM STC. SL Avda, De Los Rosales 32, 28935 Mostoless Madrid, Spain
----	-----	--



**SINGLE PATIENT USE**