

APPLE VISION BRAND CONTACT LENS-CARE PRODUCTS

1	AIRE REFRESHING DROPS
2	ENCORE COMFORT DROPS

PATIENT PRODUCT INSTRUCTIONS FOR USE & SAFETY INFORMATION

PLEASE READ THE FOLLOWING INFORMATION CAREFULLY AND KEEP IT FOR REFERENCE.

1	AIRE REFRESHING DROPS is manufactured in an ISO 13485 certified facility. This medical device is manufactured under strict GMP conditions and guarantees our customers total quality.
2	ENCORE COMFORT DROPS is manufactured in an ISO 13485 certified facility. This medical device is manufactured under strict GMP conditions and guarantees our customers total quality.

PRODUCT DESCRIPTION

1	AIRE REFRESHING DROPS is a sterile, isotonic solution that contains Sodium Hyaluronate (0.3% in w/v), Sodium Chloride, Anhydrous Disodium Hydrogen Phosphate, Sodium Dihydrogen Phosphate Dihydrate & Polyhexanide.
2	ENCORE COMFORT DROPS is a sterile, isotonic solution that contains Hydroxy Propyl Methyl Cellulose, Sodium Chloride, Boric Acid, Potassium Chloride & Sodium Per Borate.

INTENDED USE

1	AIRE REFRESHING DROPS helps to lubricate and moisturize your contact lenses while on the eyes, thus soothes and refreshes your eyes for long lasting comfortable lens wear. It is recommended for daily wear and frequent replacement contact lenses.
2	ENCORE COMFORT DROPS helps to lubricate and moisturize your contact lenses while on the eyes, thus soothes and refreshes your eyes for long lasting comfortable lens wear. It is recommended for daily wear and frequent replacement contact lenses.

PRECAUTIONS

- Do not use if you are allergic to any ingredients in this product.
- Do not use concurrently with any eye medication except on medical advice.
- To avoid contamination, do not touch the container's dropper tip to the eye or any surface.
- Once opened, discard any unused solution after 30 days.
- Store at room temperature (5°C - 30°C) and away from direct sunlight.
- Do not use **AIRE REFRESHING DROPS** / **ENCORE COMFORT DROPS** if safety ring on bottle is broken or missing.
- DO NOT MIX WITH OTHER FLUIDS EXCEPT AS DIRECTED.
- Keep the bottle tightly closed when not in use. Use before expiry date.
- Follow the daily lens care procedure as recommended by your eye care practitioner.
- Keep out of reach of children.

DIRECTIONS FOR USE

- Twist the cap to break open the safety ring. Apply 2 – 3 drops whenever needed. Blink several times. Screw the cap tightly after use.
- If you experience any eye discomfort, irritation, pain, redness or blurred vision, remove the lenses and consult your eye care practitioner IMMEDIATELY.

MARKET AVAILABILITY

- **AIRE REFRESHING DROPS** 15 ml
- **ENCORE COMFORT DROPS** 10 ml

MANUFACTURED BY

STERICON PHARMA PTE. LTD.

No.9R, Sub Layout of Plot No.9, 1st Phase, Bommasandra Industrial Area, Bangalore – 560099 India
An ISO 13485:2003 Certified Company

MANUFACTURED FOR & DISTRIBUTED BY

APPLE VISION SDN. BHD.

No.23, Jalan 6/33B, Batu 6 ½ Jalan Kepong, 52000 Kuala Lumpur, Malaysia.

www.apple-vision.com

EC	REP	MYM STC. SL Avda, De Los Rosales 32, 28935 Mostoles Madrid, Spain
----	-----	-------------------------------------------------------------------



SINGLE PATIENT USE